

Local Export – CD/DVD Creator


Local Export is used to create CD/DVD or DICOM/iSite files on the local device.

Selecting Files

Drag and drop exams with images to the Local Export folder.

Only exams with images can be selected

Click on 'Local Export' once all exams have been selected and placed

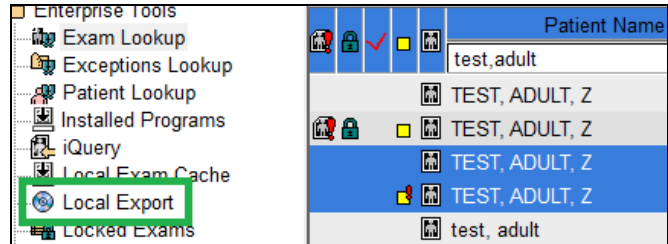


Create Files

Select the target media or file/folder with the 'Export to:' drop down.


Select the format of the image files

- 'Philips Proprietary' files will include the result and measurements
- DICOM format will allow the exam(s) to be uploaded to another PACS

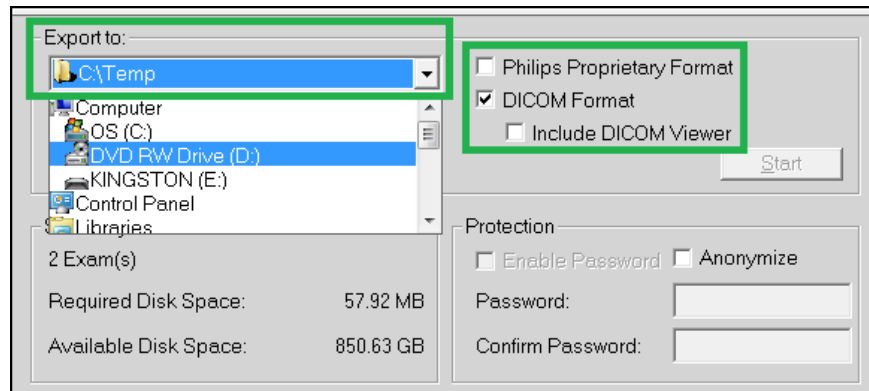

The files can be password protected if necessary

Click 'Start' to begin writing the files

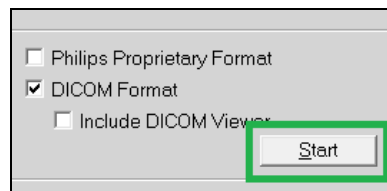

Status bar will show progress.

Facilities with slow network connections may encounter long download times

"Files exported successfully..." will appear when the files have been written

