


Saving Presentation States


In order to retain the sequence/order of images and series, annotations, WW/WL, and markings, users must 'Save a Presentation State' on the exam. This action is often taken by technologists to put the series in the correct sequence per the radiologist protocols. Some physicians will Save a Presentation State to preserve their pre-op measurements.

Save Presentation State:

With the study open, arrange the series/images, make annotations, etc. Then right-click on the Exam Margin (Gray Bar at the left end of the images/thumbnails), select 'Presentation States', and select 'Save Presentation States'.

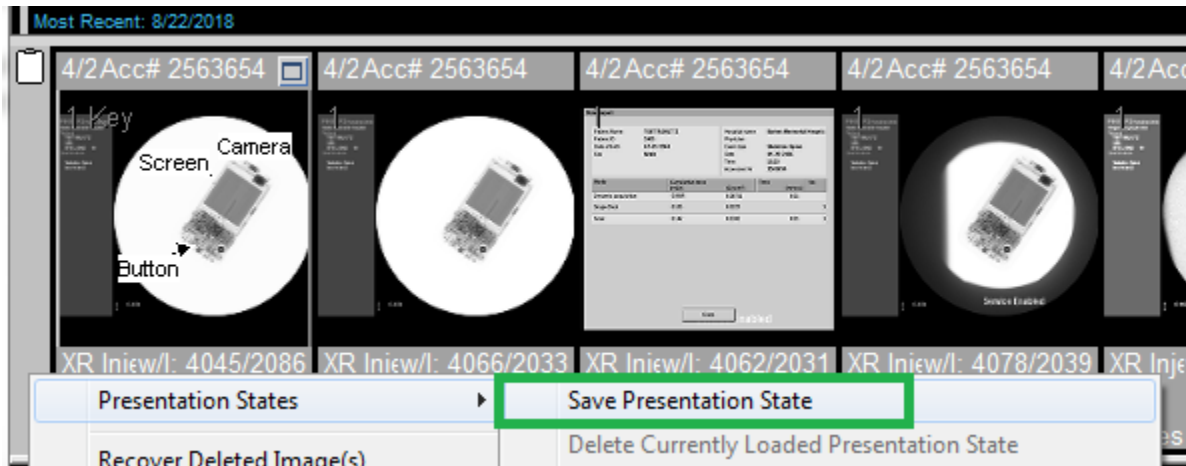

A name for the presentation state must be entered, initials are the typical name technologists use. Select the appropriate type of Presentation State and Click 'OK'.

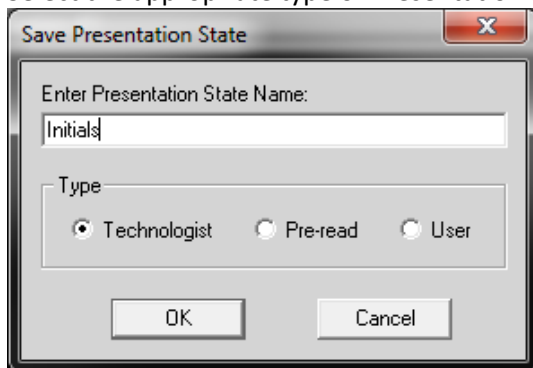

Load Presentation State:

Presentation States saved previously can be loaded by right-click on the Exam Margin, selecting 'Presentation States', selecting the presentation state from the list

